

WI-FI vs. NON WI-FI ELECTRONIC LOCKING SYSTEMS

Non Wi-Fi Elock

AVAILABLE FEATURES


- Add/Delete Users
- Change Auto Relock Timer - **150 Series Locks ONLY**
(Default is 10 mins, can be changed to 1 sec – Infinite time)

ALL FUNCTIONS MUST BE PHYSICALLY PERFORMED AT EACH CART

8 min to make changes at each cart using lock keypad
2 min travel time between carts

Ideal for customers that:

- Simply need a cart that automatically relocks
- Have a small fleet of carts
- Carts are located near each other
- Have one access code that will be shared amongst staff
- Do NOT need to track user access via audit trails
- Do NOT need to change cart settings or ADD/DELETE users very often
- Need the most economical electronic lock solution

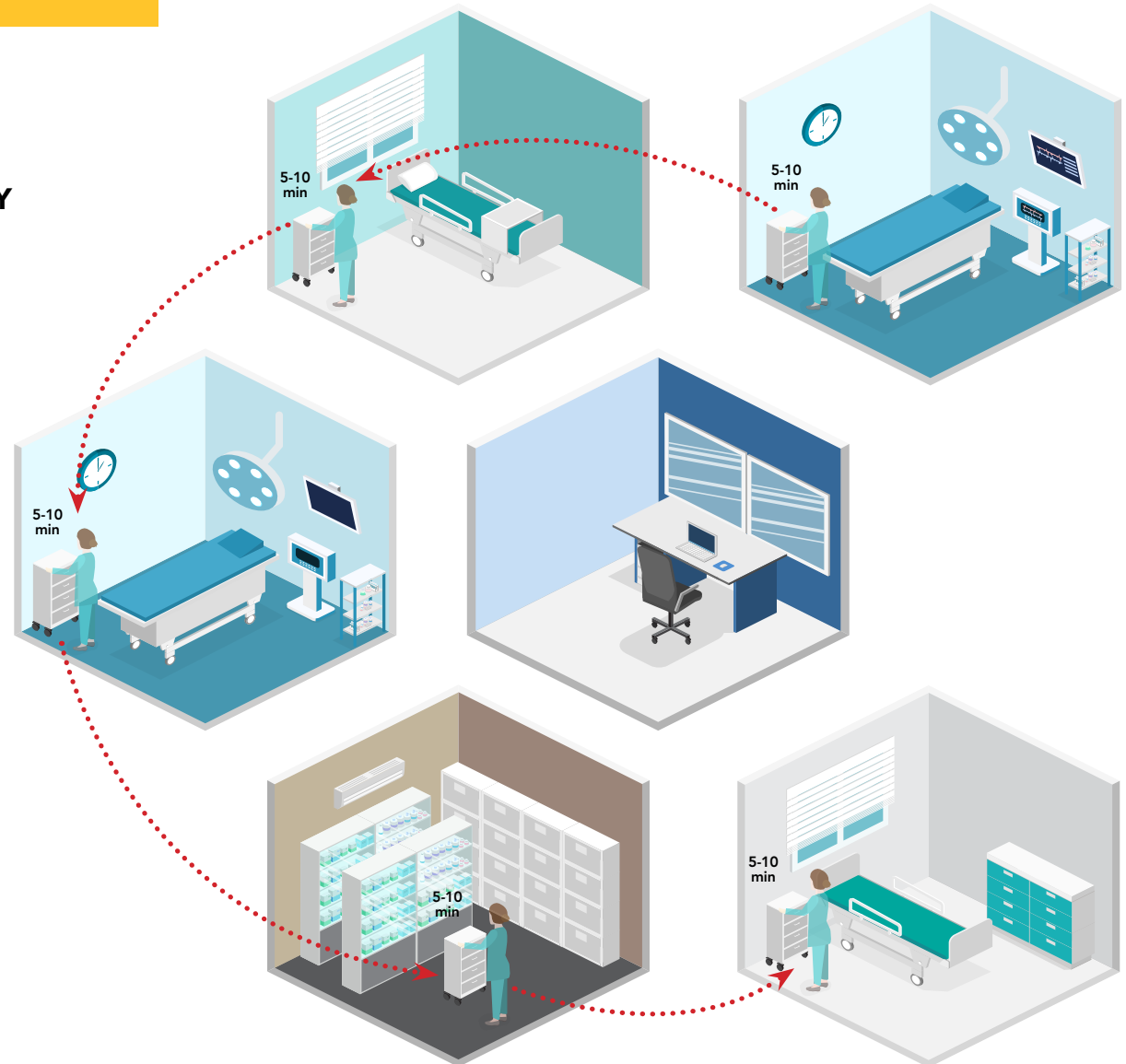

Non Wi-Fi Elock with LockManager


AVAILABLE FEATURES

- Add/Delete Users
- Change Auto Relock Timer
(Default is 10 mins, can be changed to 1 sec – Infinite time)
- Access to Audit Trails (Report of all cart access history)

FUNCTIONS CAN BE EXECUTED IN
LOCKMANAGER FIRST BUT THE DATA MUST
BE UPLOADED PHYSICALLY AT EACH CART
20 min to make changes to LockManager Software
Database on Laptop
2 min upload at each cart
2 min travel time between carts

Ideal for customers that:

- Need a cart that automatically relocks
- Have a medium sized fleet of carts
- Carts are relatively near each other
- May wish to assign individual access codes to each cart user
- Need to be able to track user access via audit trails
- Need to be able to change cart settings or ADD/DELETE users with some frequency
- Need an economical electronic lock solution

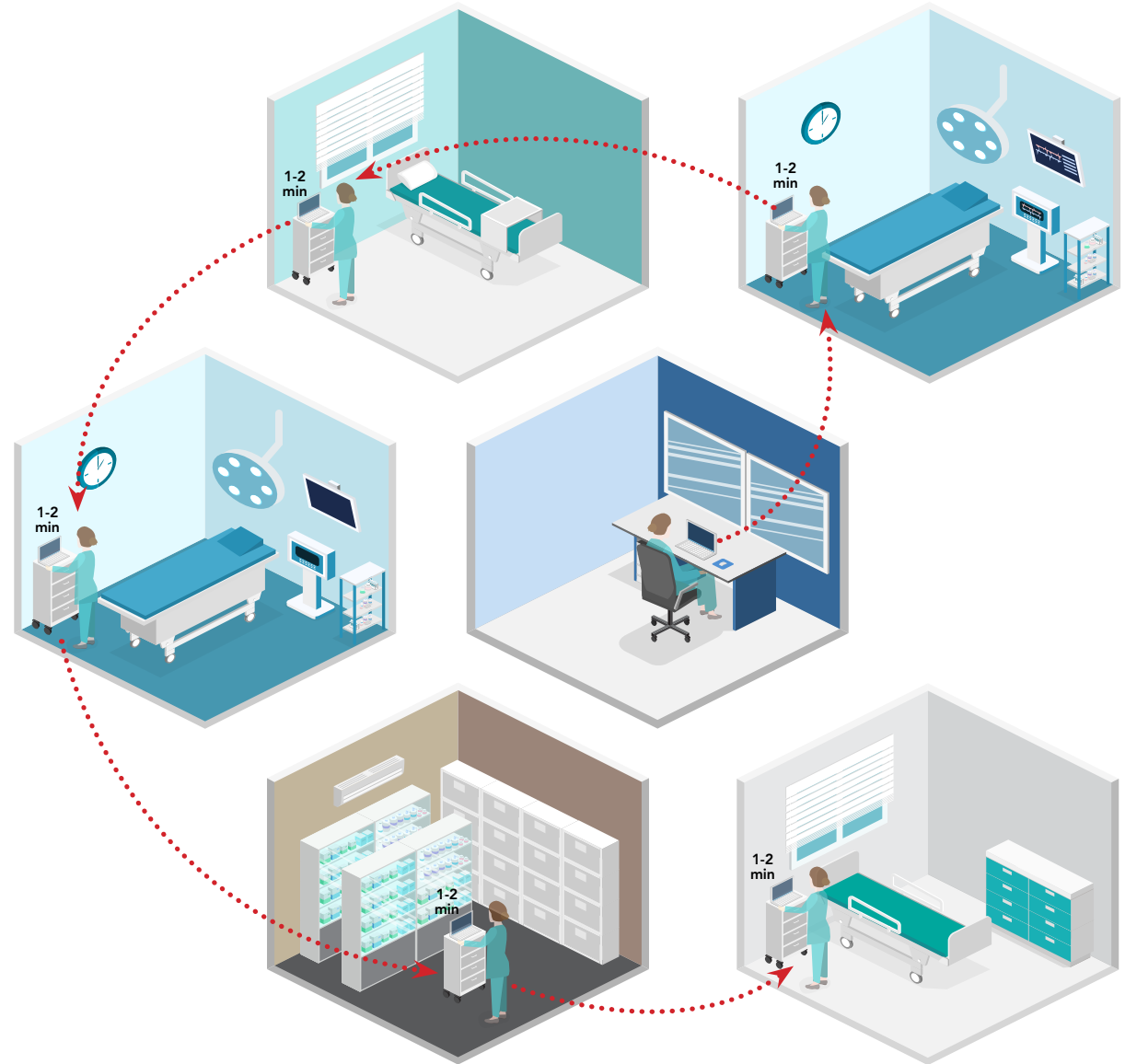

Wi-Fi Elock with LockManager

AVAILABLE FEATURES

- Add/Delete Users
- Change Auto Relock Timer
(Default is 10 mins, can be changed to 1 sec – Infinite time)
- Access to Audit Trails *(Report of all cart access history)*
- Ability to update all carts remotely from one location

ALL FUNCTIONS CAN BE EXECUTED AND
UPLOADED TO EACH CART REMOTELY VIA
LOCKMANAGER

*20 min to make changes to LockManager Software
Database on Laptop
NO travel time*

Ideal for customers that:

- Need a cart that automatically relocks
- Have a larger fleet of carts
- Carts are spread widely throughout a facility
- Wish to assign individual access codes to each cart user
- Need to be able to track user access via audit trails
- Need to be able to ADD/DELETE users and change cart settings with frequency
- Do NOT want to travel "cart to cart" to ADD/DELETE users, update lock settings or download cart access audit trails

Wi-Fi is a wireless networking technology that allows devices such as computers (laptops and desktops) and other equipment (procedure carts) to interface with the Internet.

BENEFITS OF WI-FI

Save time/more efficient • Less physical demands • Money savings long-term



TIME SAVINGS ANALYSIS

Example 1: Fleet of 10 Carts

- **Non-Wifi ELock System**
 - 8 min to make changes at each cart using lock keypad
 - 2 min travel time between carts
 - 100 minutes total to update all locks
- **Non-Wifi ELock System with LockManager**
 - 20 min to make changes to Lockmanager Software Database on Laptop
 - 2 min upload at each cart
 - 2 min travel time between carts
 - 60 minutes total to update all locks
- **Wifi ELock System with LockManager**
 - 20 min to make changes to Lockmanager Software Database on Laptop
 - NO travel time
 - 20 minutes total to update all locks

Example 1: Fleet of 100 Carts

- **Non-Wifi ELock System**
 - 8 min to make changes at each cart using lock keypad
 - 2 min travel time between carts
 - 1000 minutes total to update all locks
- **Non-Wifi ELock System with LockManager**
 - 20 min to make changes to Lockmanager Software Database on Laptop
 - 2 min upload at each cart
 - 2 min travel time between carts
 - 420 minutes total to update all locks
- **Wifi ELock System with LockManager**
 - 20 min to make changes to Lockmanager Software Database on Laptop
 - NO travel time
 - 20 minutes total to update all locks